

Medifil® II

Collagen Particles



Description

Medifil® II is 100% non-hydrolyzed type 1 bovine native collagen. Human BioSciences, Inc.'s proprietary Kollagen™ technology process protects and retains significantly more native triple helical protein structure, thus allowing superior stability of the molecule and scaffolding through all four phases of wound healing.

Indications

For use and management of *burns, scrapes, sores, ulcers, acute and chronic wounds, superficial, partial and full thickness wounds, infected and non-infected wounds, and minimal to heavily exuding wounds*. Easy application and product compatibility with topical agents and or dressings.

Application

Cleanse the wound per physician order. Apply Medifil® II particles ¼ inch to entire wound bed surface, including any undermining and or tunneling. Cover the wound with a secondary absorbent dressing to promote thermoregulation and maintain an optimal moist wound healing environment. Frequency of dressing change is based on exudate amount, clinical findings, and per physician recommendations.

Contraindications

- ▲ Known bovine sensitivities.
- ▲ Third-degree burns



- 👤 Single-patient use
- 🧼 Sterile until opened
- 🌡️ Store at room temperature
- 🚫 Do not use if package is damaged

Manufactured in the USA
Human BioSciences, Inc.
940 Clopper Rd. Gaithersburg, MD 20878 USA
info@humanbiosciences.com
+1 301-740-1893
©2020