



COLLATEK® COLLAGEN GEL

Collatek® is 100% non-hydrolyzed type 1 bovine native collagen. Human BioSciences, Inc.'s proprietary Kollagen™ technology process protects and retains significantly more native triple helical protein structure, thus allowing superior stability of the molecule and scaffolding through all four phases of wound healing.

AVAILABLE SIZES

1 oz tube

HCPCS CODE

A6011

**Products are reimbursable under Medicare in the USA.*

INDICATIONS

- Acute/Chronic wounds
- Minimal - moderate draining wounds
- Abrasions, cuts
- Skin tears
- Trauma wounds
- Burns (First and Second-degree)
- Donor sites
- Pressure Ulcers (Stage 2,3,4)
- Diabetic Ulcers
- Vascular Ulcers

APPLICATION

- Cleanse the wound with a noncytotoxic wound cleanser.
- Apply Collatek® II collagen gel generously to the wound bed surface, including in any undermining and or tunneling.
- Cover the wound with a secondary absorbent dressing to provide an advantageous healing environment.
- Change the dressing every day.

CONTRAINDICATIONS

- Known bovine sensitivities
- Third-degree burns
- Active vasculitis

PRECAUTIONS

- Single-patient use
- Sterile until opened
- Store at room temperature
- Do not use if package is damaged.



Manufactured in the USA

Human BioSciences, Inc.

940 Clopper Rd. Gaithersburg, MD 20878 USA

www.humanbiosciences.com

info@humanbiosciences.com | +1 301-740-1893

©2021

