

COLLECULE LpH

SURGICAL COLLAGEN POWDER

INDICATION FOR USE:

- Partial and full thickness wounds
- Pressure (Stage I-IV) and venous ulcers
- Ulcers caused by mixed vascular etiologies
- Venous stasis and diabetic ulcers
- 1st and 2nd degree burns
- Cuts, abrasions and surgical wounds.

PRODUCT APPLICATION:

1. Debride and cleanse the wound.
2. Apply medication to the wound if indicated.
3. Apply ColleCule LpH Surgical Powder 1/4" thick to the wound surface. Do not pack tightly, allow for expansion of particles.
4. Cover with secondary absorbent dressing.

PRODUCT REMOVAL:

1. Change as indicated by the amount of drainage and instructions provided with the absorbent cover dressing.
2. Flush with Saline solution and repeat steps under product application as needed.

PRECAUTIONS:

1. Do not use ColleCule LpH on patients with bovine sensitivities.
2. Do not use on patients with previous skin reactions to hydrocolloid products.
3. Discontinue use if excessive redness, pain, swelling or blistering occurs.
4. Avoid eye contact and flush thoroughly with water if this occurs.
5. This product is single use only. All open and unused product must be discarded.

STORAGE:

Store at temperatures less than 90F or 32C.

Manufactured By:

Human Biosciences Inc.

600 N Queen St, Martinsburg, WV 25404

www.humanbiosciences.com | Email: info@humanbiosciences.com | Tel: +1 301-740-1893

Product code: CSP1001, CSP1050 | Rev. 2, May 2026

STERILE R

Method of sterilization: Irradiation



Do not reuse



Do not resterilize



Keep dry



Do not use if package is damaged



Keep away from sunlight



Not made with natural rubber latex



Temperature limitation

Rx Only

Prescription use only



Instruction for use in electronic form

