

COLLECULE LpH

COLLAGEN PAD 4" X 5.25"

INDICATION FOR USE:

- This device is intended for the management of burns, sores, blisters, scrapes, ulcers and other wounds.

PRODUCT APPLICATION:

- Cleanse the wound.
- Apply ColleCule LpH™ Collagen Pad on the wound. For dry wounds, moisten with saline.
- Cover with an absorbent secondary dressing.
- Continue use until the wound is healed.
- Maintain a moist wound environment.

PRODUCT REMOVAL:

- Change dressing as indicated by drainage and cover dressing instructions.
- After removing the cover dressing, repeat steps under Product Application as needed.

PRECAUTIONS:

- Do not use on patients with sensitivities to bovine materials.
- Discontinue use and contact a healthcare provider if redness, pain, swelling, or blistering occurs.
- Do not reuse; risk of contamination or infection.
- Single-use only. Discard all open or unused pouches.
- Avoid eye contact; flush with water if contact occurs.

STORAGE:

Store in a cool, dry place 50°F – 80°F (10°C – 27°C).
Sterility guaranteed only in unopened, undamaged packages.

Manufactured By:

Human Biosciences Inc.

600 N Queen St, Martinsburg, WV 25404

www.humanbiosciences.com | Email: info@humanbiosciences.com | Tel: +1 301-740-1893

Product code: CSP1045 | Rev. 2, May 2026

STERILE R

Method of sterilization: Irradiation



Do not reuse



Do not resterilize



Keep dry



Do not use if package is damaged



Keep away from sunlight



Not made with natural rubber latex



Temperature limitation

Rx Only

Prescription use only

