



MEDIFIL® II

COLLAGEN PARTICLES

Medifil® II is 100% non-hydrolyzed type 1 bovine collagen. Human BioSciences, Inc.'s proprietary Kollagen™ technology process protects and retains significantly more native triple helical protein structure, thus allowing superior stability of the molecule and scaffolding through all four phases of wound healing.

AVAILABLE SIZES

HCPCS CODE

1 gram vial

A6010

1 gram pouch

A6010

**Products are reimbursable under Medicare in the USA.*

INDICATIONS

The device is intended for the management of burns, sores, blisters, ulcers, scrapes and other wounds.

APPLICATION

- Cleanse the wound with a noncytotoxic wound cleanser, extending to the surrounding skin.
- Apply Medifil® II particles ¼ inch to the entire wound bed surface, including any deeper areas.
- Cover the wound with an absorbent dressing to provide an advantageous healing environment.
- Change the dressing based on the drainage amount.

CONTRAINDICATIONS

- Known bovine sensitivities
- Third-degree burns
- Active vasculitis

PRECAUTIONS

- Single-patient use
- Sterile until opened
- Store at room temperature
- Do not use if package is damaged



Manufactured in the USA

Human BioSciences, Inc.

600 N Queen St, Martinsburg, WV 25404

www.humanbiosciences.com

info@humanbiosciences.com | +1 301-740-1893

©2026

