



KOLLAVET® ADVANCED WOUND CARE COLLAGEN GEL

Safe & Natural BioActive Triple Helical Collagen

Kollavet® Collagen gel is the next generation of medical-grade higher nativity of collagen for healing minor or major animal wounds. Our collagen products are designed to help animal wounds heal quickly and naturally. Kollavet gel is viscous and keeps its shape, allowing it to be used to donate moisture and as a wound filler. Kollavet products are safe, natural, and effective.

AVAILABLE SIZES

1 oz. tube (30 gram)

INDICATIONS

- Small to large draining wounds.
- Surgical sites, traumatic wounds, superficial wounds, burns, footpad injuries, hot spots, and lick granulomas.

INFORMATION

- This device is intended for single use only.
- All open and unused vials must be discarded.
- Store in a cool and dry place.

APPLICATION


- Cleanse the wound with a wound cleanser.
- Apply Kollavet® gel generously to the entire wound surface.
- Cover with an absorbent dressing or aerosol bandage.
- Use Kollavet® gel until the wound is fully healed.

REMOVAL

- Change the dressing every day.
- After removal of the cover dressing, repeat steps under Product Application as needed.

PRECAUTIONS

- This product is made from bovine (cattle) derived material.
- All open or unused products must be discarded.
- Store at room temperature.

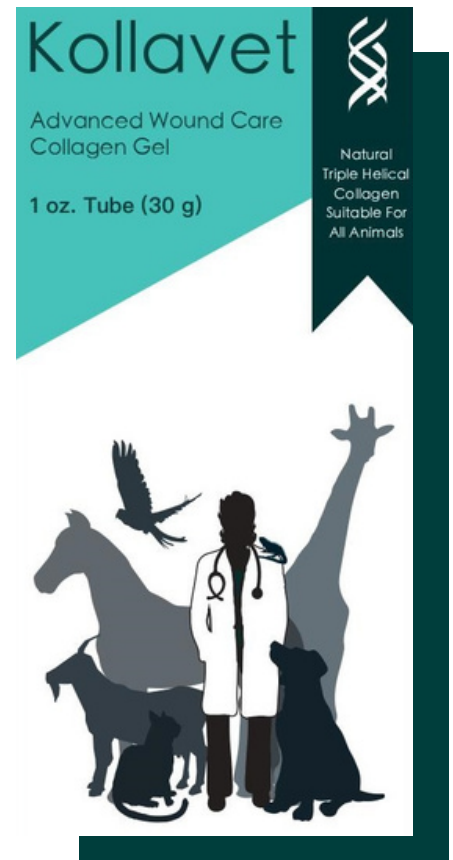

Manufactured in the USA

Human BioSciences, Inc.

940 Clopper Rd. Gaithersburg, MD 20878 USA

www.humanbiosciences.com

info@humanbiosciences.com | +1 301-740-1893

©2021

