



KOLLAVET® ADVANCED WOUND CARE COLLAGEN SHEETS

Safe & Natural BioActive Triple Helical Collagen

Kollavet® Collagen Sheets are the next generation of medical-grade higher nativity collagen for healing minor or major animal wounds. Our collagen products are designed to help animal wounds heal quickly and naturally. Kollavet® Collagen Sheet Dressings are easy to apply, safe, natural, and effective.

AVAILABLE SIZES

2x2, 3x4, 7x7, 8x12

INDICATIONS

- Small to large draining wounds.
- Surgical sites, traumatic wounds, superficial wounds, burns, footpad injuries, hot spots, and lick granulomas.

INFORMATION

- This device is intended for single use only.
- All open and unused vials must be discarded.
- Store in a cool and dry place.

APPLICATION


- Cleanse the wound with a wound cleanser.
- Apply Kollavet® sheet to the entire wound surface.
- Cover with an absorbent dressing or aerosol bandage.
- Use Kollavet® sheet until the wound is fully healed.

REMOVAL

- Change the dressing every day.
- After removal of the cover dressing, repeat steps under Product Application as needed.

PRECAUTIONS

- This product is made from bovine (cattle) derived material.
- All open or unused products must be discarded.
- Store at room temperature.

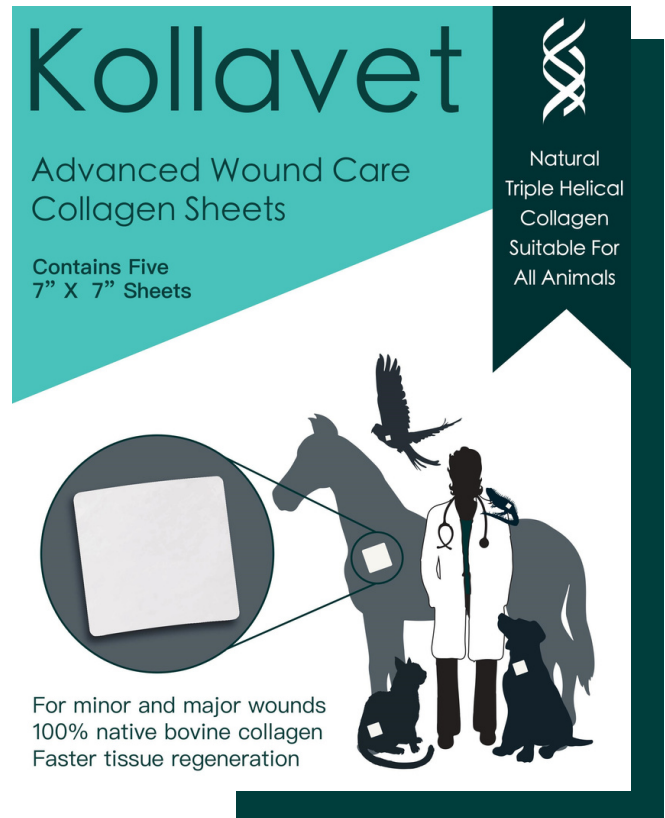

Manufactured in the USA

Human BioSciences, Inc.

940 Clopper Rd. Gaithersburg, MD 20878 USA

www.humanbiosciences.com

info@humanbiosciences.com | +1 301-740-1893

©2021

