

Medifil® II

Collagen Particles (1g)

Indication:

This device is intended for the management of burns, sores, blisters, scrapes, ulcers and other wounds.

Product Application:

1. Cleanse the wound per physician order.
2. Apply Medifil® II particles ¼ inch thick to the entire wound surface.
Do not pack tightly; allow for expansion of particles.
3. Cover with an absorbent dressing.
4. Use Medifil® II product until the wound is fully healed.
5. Maintain a moist wound environment.

Product Removal:

1. Change as indicated by the amount of drainage and instructions provided with the absorbent cover dressing.
2. After removal of the cover dressing, repeat steps under Product Application as needed.

Precautions:

- This product is made from bovine (cattle) derived materials.
Do NOT use this product if bovine-derived product such as leather cause an allergy reaction or sensitivity upon contact with the patient's skin.
- WARNING: Discontinue use of Medifil® II particles and notify your doctor if excessive redness, pain, swelling or blistering occurs.
- WARNING: Potential risk of loss of sterility and infection if opened and reused.
- This device is intended for single use only. All open and unused vials must be discarded.
- If sensitivity develops, discontinue use.

Manufactured By:

Human Biosciences Inc.

600 N Queen St, Martinsburg, WV 25404

www.humanbiosciences.com | Email: info@humanbiosciences.com | Tel: +1 301-740-1893

HCPC Code: A6010 | Product code: MF2001 and MF2002 | Rev 2, April 2026

STERILE R

Method of sterilization: Irradiation



Do not reuse



Do not resterilize



Keep dry



Do not use if package is damaged



Keep away from sunlight



Not made with natural rubber latex



Temperature limitation

Rx Only

Prescription use only



Instruction for use in electronic form

