



# MEDIFIL® II

## COLLAGEN PARTICLES

Medifil® II is 100% non-hydrolyzed type 1 bovine collagen. Human BioSciences, Inc.'s proprietary Kollagen™ technology process protects and retains significantly more native triple helical protein structure, thus allowing superior stability of the molecule and scaffolding through all four phases of wound healing.

### AVAILABLE SIZES

- 1 gram vial
- 1 gram pouch

### HCPCS CODE

- A6010
- A6010

*\*Products are reimbursable under Medicare in the USA.*

### INDICATIONS

- Acute/Chronic wounds
- Trauma wounds
- Abrasions
- Skin tears
- Moisture lesions
- Burns (First and Second-degree)
- Donor sites
- Pressure Ulcers (Stage 2,3,4)
- Diabetic Ulcers
- Vascular Ulcers



### APPLICATION

- Cleanse the wound with a noncytotoxic wound cleanser, extending to the surrounding skin.
- Apply Medifil® II particles ¼ inch to the entire wound bed surface, including any deeper areas.
- Cover the wound with an absorbent dressing to provide an advantageous healing environment.
- Change the dressing based on the drainage amount.

### CONTRAINDICATIONS

- Known bovine sensitivities
- Third-degree burns
- Active vasculitis

### PRECAUTIONS

- Single-patient use
- Sterile until opened
- Store at room temperature
- Do not use if package is damaged

Manufactured in the USA

**Human BioSciences, Inc.**

940 Clopper Rd. Gaithersburg, MD 20878 USA

[www.humanbiosciences.com](http://www.humanbiosciences.com)

[info@humanbiosciences.com](mailto:info@humanbiosciences.com) | +1 301-740-1893

©2021

